

# AESCULAP Aicon® – Label All-In-ONE<sup>1</sup>


## REMOVABLE LABEL


### 1 BEFORE STERILISATION



Affix the label


Which protective paper is removed for which packaging type?

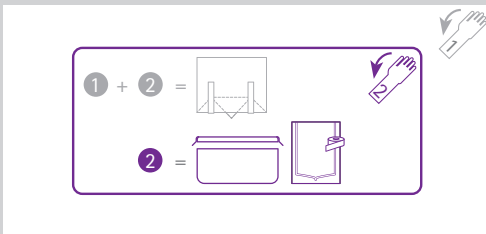

Application for anodized aluminium surfaces and sealable transparent bag (for compound plastic foil) remove the protective paper 



Application for sterilisation fleece, before use remove the protective paper completely  + 

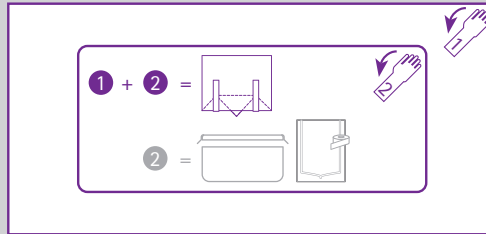

### 2 AFTER STERILISATION

Remove the label from the sterile packaging for permanent documentation into the patient file

### 3 FOR DOCUMENTATION

Affix the label in the patient file.

REMOVE the protective paper  +  and the interlayer film COMPLETELY 



#### Label with process indicator for the steam sterilisation

Description of the color change

Before sterilisation	After sterilisation
blue	brown

■ Process indicator complies with the ISO 11140-1 Type 1

#### Specifications

- Label size 75 x 35 mm
- CE – marking
- AESCULAP® thermo transfer ribbons:

IC-V008 suitable for the industrial printer

IC-V014 suitable for the desktop printer

outer print in black colour<sup>2</sup>

#### AESCULAP® – Label All In ONE

Flexible use for different surfaces:

- anodized aluminium
- sealable transparent bag (on the film surface)
- sterilisation fleece
- paper for the documentation in patient file

D-ST19058 0720/PDF/2

<sup>1</sup>For steam sterilisation process only | <sup>2</sup>Product information can be requested by your Aesculap contact person.

AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 7461 95-0 | Fax +49 7461 95-26 00 | www.aesculap.com